Instructions for Use

Hepcidin 25 (bioactive) HS ELISA

IVD

CE

REF EIA-5782

Σ 96

DRG Instruments GmbH, Germany
Frauenbergstraße 18, 35039 Marburg
Phone: +49 (0)6421-1700 0, Fax: +49 (0)6421-1700 50
Website: www.drg-diagnostics.de
E-mail: drg@drg-diagnostics.de

Distributed by:

DRG International, Inc., USA
841 Mountain Ave., Springfield, NJ 07081
Phone: (973) 564-7555, Fax: (973) 564-7556
Website: www.drg-international.com
E-mail: corp@drg-international.com

Please use only the valid version of the Instructions for Use provided with the kit.
Verwenden Sie nur die jeweils gültige, im Testkit enthaltene, Gebrauchsanweisung.
Si prega di usare la versione valida delle istruzioni per l'uso a disposizione con il kit.
Por favor, use sólo la versión válida de las instrucciones de uso que se suministran con el kit.
Utilisez seulement la version valide des Instructions d'utilisation fournies avec le kit.
Utilize apenas a versão válida das Instruções de Utilização fornecidas com o kit.

Introduced modifications / Durchgeführte Änderungen / Modifiche introdotte / Modificaciones introducidas / Modifications apportées / Modificações introduzidas

The following changes have been made in comparison to the previous version:

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Detailed editorial revision. In general, the IFU is updated according to the requirements of the IVD Regulation (EU) 2017/746. Changed wording in several chapters.

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1 INTENDED USE:	More definitions for intended use and use not intended.
4.1 Materials provided with the kit:	Concentration for Stop Solution is given now in percent instead of mol/L ("< 5 % H ₂ SO ₄ " instead of "0,5 M H ₂ SO ₄ ")
4.4 Reagent Preparation:	Correction for diluted Wash Solution: Stability is 1 week at RT (before: 2 weeks);
5 SAMPLE COLLECTION, STORAGE AND PREPARATION	Added: "Lithium heparin plasma may lead to slightly decreased values (on average by 15%)."
9.5 Linearity:	Updated values for all plasma samples.
10.1 Interfering Substances:	Added: "A biotin concentration of up to 1200 ng/mL in a sample has no influence on the assay results."

ENGLISH	2
DEUTSCH	11
ITALIANO	18
ESPAÑOL	25
FRANÇAIS	31
LITERATURE	38
SYMBOLS USED	39

1 INTENDED USE

The DRG Hepcidin 25 (bioactive) HS ELISA is a manual enzyme immunoassay for the **quantitative** measurement of Hepcidin-25 in human serum or plasma (EDTA, Li-heparin or citrate plasma).

For *in vitro* diagnostic use. For laboratory professional use.

The device is intended to be used as an aid to diagnosis for individuals where information on one or more of the following is required:

- iron homoeostasis
- iron deficiency
- anemia
- erythropoiesis
- hemochromatosis

The device is **not** intended for evaluation of hepatitis C and liver cirrhosis.

1.1 Scientific Validity

Hepcidin is an iron homoeostasis regulator peptide. The bioactive peptide Hepcidin-25 is generated predominantly in the liver by proteolytic cleavage of the C-terminal 25 amino acids of prohepcidin (1). Subsequent N-terminal processing of Hepcidin 25 results in smaller peptides of 20-24 amino acids that show greatly reduced activity and accumulate in the urine (2).

Although originally identified as antimicrobial peptide (3), Hepcidin-25 is now established as a major regulator of dietary iron absorption and cellular iron release (4). Hepcidin exerts its regulatory function by counteracting the function of ferroportin, the major cellular iron exporter in the membrane of macrophages, hepatocytes, and the basolateral site of enterocytes. Hepcidin 25 induces the internalization and degradation of ferroportin, resulting in increased intracellular iron stores, decreased dietary iron absorption, and decreased circulating iron concentrations (5).

Hepatocellular hepcidin synthesis decreases under conditions of increased demand for circulating iron like iron deficiency, hypoxia, anemia, and erythropoiesis. In contrast, hepcidin synthesis is induced by inflammation and infection (6).

Serum Hepcidin-25 has been shown to add value to identify and differentiate specific disease conditions. Hepcidin deficiency causes hereditary hemochromatosis, characterized by body iron overload that may progress to liver cirrhosis (7). In addition, low Hepcidin-25 concentration can be induced by iron loading anemias (8) and chronic hepatitis C (9). In contrast, high Hepcidin-25 levels have been found in iron-refractory iron-deficiency anemia (10), during infection, chronic kidney disease (11), and after intensive exercise, explaining the high iron deficiency among athletes (12).

Product Patent Protected:

Pat. US 7,320,894 B2; US 8,017,737 B2, EP 2109624, EP 1578254,

Japan 4638350, Russia 2 359 268 C2, China 200380108964.8, Hong Kong 1114419, Canada 2,506,668a

2 PRINCIPLE OF THE TEST

The DRG Hepcidin 25 (bioactive) HS ELISA is a solid phase enzyme-linked immunosorbent assay (ELISA) based on the **principle of competitive binding**.

The microtiter wells are coated with a monoclonal antibody (mouse) directed towards a unique antigenic site of the Hepcidin-25 molecule.

Endogenous Hepcidin-25 in a sample competes with a Hepcidin-25-biotin conjugate (*Enzyme Conjugate*) for binding to the coated antibody.

After incubation the microtiter plate is washed to stop the competition reaction. In the following incubation the bound biotin molecules are detected with streptavidin peroxidase (*Enzyme Complex*).

After a washing step to remove all unbound substances, the solid phase is incubated with the substrate solution. The colorimetric reaction is stopped by addition of stop solution, and optical density (OD) of the resulting yellow product is measured. The intensity of color is inversely proportional to the concentration of the analyte in the sample.

A standard curve is constructed by plotting OD values against concentrations of standards, and concentrations of unknown samples are determined using this standard curve.

3 WARNINGS AND PRECAUTIONS

- This kit is for *in vitro* diagnostic use only. For laboratory professional use only.
- Before starting the assay, read the instructions for use completely and carefully. Use the valid version of instructions for use provided with the kit. Be sure that everything is understood.
- Do not mix or use components from kits with different lot numbers. It is advised not to interchange wells of different plates even of the same lot. The kits may have been shipped or stored under different conditions and the binding characteristics of the plates may result slightly different.
- Do not use reagents beyond expiry date as shown on the kit labels.
- Do not reuse microtiter wells.
- Reagents of other manufacturers must not be used together with the reagents of this test kit.
- All reagents in this kit are clear liquids, substrate solution is clear and colorless. Changes in its appearance may affect the performance of the test. In that case, contact DRG.
- Microbial contamination of reagents or samples may give false results.
- Allow the reagents to reach room temperature (20 °C to 26 °C) before starting the test. Temperature will affect the optical density readings of the assay.
- All indicated volumes must be performed according to the protocol. Optimal test results are only obtained when using calibrated pipettes and microtiter plate readers.
- Use reservoirs only for single reagents. This especially applies to the substrate reservoirs. Using a reservoir for dispensing a substrate solution that had previously been used for the conjugate solution may turn solution coloured. Do not pour reagents back into original vials as reagent contamination may occur.

General precautions

- Follow good laboratory practice and safety guidelines.
- Never pipet by mouth and avoid contact of reagents and samples with skin and mucous membranes.
- Do not smoke, eat, drink, or apply cosmetics in areas where samples or kit reagents are handled.
- Wear lab coats and disposable latex gloves when handling samples and reagents and where necessary safety glasses.

Biohazard information

- All reagents of this test kit which contain human serum or plasma have been tested and confirmed negative for HIV I/II, HBsAg and HCV by FDA approved procedures. However, no known test method can offer total assurance that no infectious agent is present.
- The device contains material of animal origin, which is certified apparently free of infectious or contagious diseases and injurious parasites.
- Bovine components originate from countries where BSE (Bovine spongiform encephalopathy) has not been reported.
- All materials and samples of human or animal origin must be handled as if capable of transmitting infectious diseases.
- Handling must be done in accordance with the procedures defined by appropriate national biohazard and safety guideline or regulation. Waste must be discarded according to local rules and regulations.

Information to chemical hazards and hazard classification

- Some reagents contain preservatives in non-declarable concentrations. Nevertheless, in case of contact with eyes or skin, flush immediately with water.
- Substrate Solution contains an ingredient in non-declarable concentrations which causes serious eye irritation. In case of possible contact with eyes, rinse immediately carefully and thoroughly with eye wash or water. After contact with skin, wash with plenty of water. Take-off contaminated clothing and wash it before reuse.
- Avoid contact with Stop Solution containing < 5 % H₂SO₄. It may cause skin irritation and burns.
- Chemicals and prepared or used reagents must be treated as hazardous waste according to the national safety guideline or regulation.
- This product does not contain substances which have carcinogenic, mutagenic or toxic for reproduction (CMR) properties.

All reagents of this test kit do NOT contain hazardous substances in concentrations to be declared, a classification and labelling is not required.

For detailed information, please refer to the Safety Data Sheet, which is available upon request directly from DRG.

4 MATERIALS

4.1 Materials provided with the kit

Symbol	Quantity	Description	Preparation
Microtiterwells	12 x 8 wells (break apart)	Microtiter plate Coated with anti-Hepcidin-25 antibody (monoclonal).	Ready to use
Standard (Standard 0 - 5)	6 x 0.5 mL	Standards * Concentrations: 0 – 1 – 3 – 9 – 27 – 81 ng/mL Conversion: 1 ng/mL = 0.358 nmol/L	Lyophilized; See "Reagent Preparation".
Control Low & Control High	2 x 0.5 mL	Controls * <i>For control values and ranges please refer to vial label or Certificate of Analysis.</i>	Lyophilized; See "Reagent Preparation".
Sample Diluent	1 x 3 mL	Sample Diluent *	Ready to use
Enzyme Conjugate	1 x 7 mL	Enzyme Conjugate * Hepcidin-25 conjugated to biotin;	Ready to use
Enzyme Complex	1 x 14 mL	Enzyme Complex * Streptavidin conjugated to horseradish peroxidase	Ready to use
Substrate Solution	1 x 14 mL	Substrate Solution Contains 3,3',5,5'-tetramethylbenzidine (TMB). <i>Keep away from direct sun light.</i>	Ready to use
Stop Solution	1 x 14 mL	Stop Solution Contains < 5 % H ₂ SO ₄ . <i>Avoid contact with the stop solution. It may cause skin irritations and burns.</i>	Ready to use
Wash Solution	1 x 30 mL	Wash Solution, 40X concentrate *	See "Reagent Preparation".
	1 x	Instructions for Use	
	1 x	Certificate of Analysis (CoA)	
		* Contains non-mercury preservative.	

4.2 Materials required but not provided

- A calibrated microtiter plate reader (450 nm, with reference wavelength at 620 nm to 630 nm)
- Calibrated variable precision micropipettes
- Manual or automatic equipment for rinsing microtiter plate wells
- Absorbent paper
- Distilled water
- Timer
- Graph paper or software for data reduction

4.3 Storage and Stability of the Kit

Unopened kits and reagents as well as **opened reagents** must be stored at 2 °C to 8 °C.

The microtiter plate contains snap-off strips. Do not open the pouch of the wells until it reaches room temperature. Unused wells must be stored at 2 °C to 8 °C in the sealed foil pouch including the desiccant and used in the plate frame provided. Once the foil bag has been opened, care must be taken to close it tightly again.

Once opened, reagent vials must be closed tightly again.

	Storage Temperature	Stability
Unopened kits and unopened reagents	2 °C to 8 °C	Until the expiration date printed on the label. Do not use reagents beyond this date!
Opened kit	2 °C to 8 °C	8 weeks (For reconstituted reagents refer to "4.4 Reagent Preparation".)

4.4 Reagent Preparation

Bring all reagents and required number of strips to room temperature (20 °C to 26 °C) prior to use.

Standards

Reconstitute the lyophilized contents of each standard vial with 0.5 mL distilled water and let stand for at least 10 minutes at room temperature. Mix several times before use.

Stability after reconstitution:	at 2 °C to 8 °C	8 weeks
	at -20 °C (in aliquots)	For longer storage

Controls

Reconstitute the lyophilized content of each control vial with 0.5 mL distilled water and let stand for at least 10 minutes at room temperature. Mix several times before use.

Stability after reconstitution:	at 2 °C to 8 °C	8 weeks
	at -20 °C (in aliquots)	For longer storage

Wash Solution

Add distilled water to the 40X concentrated Wash Solution.

Dilute 30 mL of concentrated *Wash Solution* with 1170 mL distilled water to a final volume of 1200 mL.

Stability after dilution:	at 20 °C to 26 °C	1 week
---------------------------	-------------------	--------

4.5 Disposal of the Kit

The disposal of the kit and all used materials/reagents must be performed according to the national regulations. Special information for this product is given in the Safety Data Sheet, section 13.

4.6 Damaged Test Kits

In case of any damage to the test kit or components, DRG must be informed in writing, at the latest one week after receiving the kit. Damaged single components must not be used for a test run. They have to be stored until a final solution has been found. After this, they must be disposed of according to the official regulations.

5 SAMPLE COLLECTION, STORAGE AND PREPARATION

The following sample material can be used in this test:

Human serum or plasma (EDTA plasma, lithium heparin plasma or citrate plasma)

Lithium heparin plasma may lead to slightly decreased values (on average by 15%).

Samples containing sodium azide should not be used in the assay.

In general, it should be avoided to use hemolytic, icteric, or lipemic samples. For further information refer to chapter "*Interfering Substances*".

5.1 Sample Collection

Serum: Collect blood by venipuncture (e.g. Sarstedt Monovette for serum), allow to clot, and separate serum by centrifugation at room temperature. Do not centrifuge before complete clotting has occurred. Patients receiving anticoagulant therapy may require increased clotting time.

Plasma: Whole blood should be collected into centrifuge tubes containing anticoagulant (e.g. Sarstedt Monovette with the appropriate plasma preparation) and centrifuged immediately after collection.

Whole blood should not be frozen before centrifugation.

5.2 Samples Storage

Samples must be stored tightly capped prior to performing the assay. If stored frozen, freeze only once. Thawed samples must be inverted several times prior to testing.

Stability	at 2 °C to 8 °C	4 days
	at -20 °C (in aliquots)	up to 12 months

5.3 Sample Preparation

Samples can be assayed without additional preparation.

6 ASSAY PROCEDURE

6.1 Procedural Notes

- All reagents and samples must be allowed to come to room temperature (20 °C to 26 °C) before use.
- All reagents must be mixed without foaming.
- Do not interchange caps of reagent vials to avoid cross-contamination.
- Use new disposal plastic pipette tips for each standard, control, or sample in order to avoid carry-over.
- To avoid cross-contamination and falsely elevated results pipette patient samples and dispense conjugate without splashing accurately to the bottom of wells.
- Mix the contents of the microtiter plate wells thoroughly to ensure good test results.
- Do not let wells dry during assay; add reagents immediately after completing the rinsing steps.
- Once the test has been started, all steps must be completed without interruption and in the same sequence for each step.
- The enzymatic reaction is linearly proportional to time and temperature.
- Optical density is a function of the incubation time and temperature. Respect the incubations times and temperatures as given in chapter "Test Procedure".
- Before starting the assay, it is recommended that all reagents are ready, caps removed, all needed wells secured in holder, etc. This will ensure equal elapsed time for each pipetting step without interruption.

- Important note to wash procedure:

Washing is critical. Improperly washed wells will give erroneous results. The sensitivity and precision of this assay is markedly influenced by the correct performance of the washing procedure!

- Test performance using fully automated analysis devices:

Automated test performance using fully automated, open-system analysis devices is possible. However, the combination must be validated by the user.

6.2 Test Procedure

Each run must include a standard curve.

The controls serve as internal controls for the reliability of the test procedure. They must be assayed with each test run.

The given test procedure describes manual processing.

Important note: The accuracy of this assay is markedly influenced by the correct incubation temperature.

1. Secure the desired number of microtiter wells in the frame holder.
2. Pipette **20 µL** of each **Standard, Control, and sample** with new disposable tips into appropriate wells.
3. Add **50 µL Enzyme Conjugate** into each well.
Thoroughly mix for 10 seconds. It is important to have a complete mixing in this step.
4. Incubate for **60 minutes** at room temperature.
5. Wash the wells as follows:
If the wash step is performed manually:
Briskly shake out the contents of the wells.
Rinse the wells **4 times** with **300 µL** diluted *Wash Solution* per well.
If an automated plate washer is used:
Rinse the wells **4 times** with **400 µL** diluted *Wash Solution* per well.
At the end of the washing step, always strike the wells sharply on absorbent paper to remove residual droplets!
6. Pipette **100 µL Enzyme Complex** into each well.
7. Incubate for **30 minutes** at room temperature.
8. Wash the wells as described in step 5.
9. Pipette **100 µL** of **Substrate Solution** to each well.
10. Incubate for **20 minutes** at room temperature.
11. Stop the enzymatic reaction by adding **100 µL** of **Stop Solution** to each well.
12. Measure the optical density (OD) of the solution in each well at **450 nm (reading)** and at **620 nm to 630 nm (background subtraction, recommended)** with a microtiter plate reader.
It is recommended that the wells be read **within 10 minutes** after adding the *Stop Solution*.

6.3 Calculation of Results

1. The concentration of the samples can be read directly from the standard curve.
2. For duplicate determinations, the mean of the two optical density (OD) values for each standard, control, and patient sample must be taken. If the two values deviate substantially from one another, DRG recommends retesting the samples.
3. Samples with concentrations exceeding the highest standard can be further diluted with *Sample Diluent* and re-assayed as described in "Test Procedure", or must be reported as > 81 ng/mL. For the calculation of the concentrations, this dilution factor must be considered.
(Example: a) dilution 1:10: 10 µL sample + 90 µL *Sample Diluent* (mix thoroughly)
b) dilution 1:100: 10 µL dilution a) 1:10 + 90 µL *Sample Diluent* (mix thoroughly).)
4. Automated method:
The results in the instructions for use have been calculated automatically using a four-parameter logistic (4PL) curve fit.
(4PL Rodbard or 4PL Marquardt are the preferred methods.) Other data reduction functions may give slightly different results.
5. Manual method:
Using semi-logarithmic graph paper, construct a standard curve by plotting the (mean) OD obtained from each standard against its concentration with OD value on the vertical (Y) axis and concentration on the horizontal (X) axis.
Determine the corresponding sample concentration from the standard curve by using the (mean) OD value for each sample.

6.3.1 Example of Typical Standard Curve

The following data is for demonstration only and **cannot** be used in place of data generations at the time of assay.

Standard	Optical Density (450 nm)
Standard 0 (0 ng/mL)	2.03
Standard 1 (1 ng/mL)	1.70
Standard 2 (3 ng/mL)	1.29
Standard 3 (9 ng/mL)	0.77
Standard 4 (27 ng/mL)	0.37
Standard 5 (81 ng/mL)	0.17

7 REFERENCE VALUES

It is strongly recommended that each laboratory determine its own reference values.

In a study conducted with apparently healthy adults, using the DRG Hepcidin 25 (bioactive) HS ELISA the following data were observed:

Population	n	Range (min. – max.) (ng/mL)	Mean (ng/mL)	2.5 th - 97.5 th Percentile (ng/mL)	Median (ng/mL)
Males and Females	75	0.25 - 47.66	16.45	1.49 - 41.46	13.47

The results alone should not be the only reason for any therapeutic consequences. The results must be correlated to other clinical observations and diagnostic tests.

8 QUALITY CONTROL

Good laboratory practice requires that controls be run with each calibration curve. A statistically significant number of controls should be assayed to establish mean values and acceptable ranges to assure proper performance.

It is recommended to use control samples according to state and federal regulations. The use of control samples is advised to assure the day-to-day validity of results. Use controls at both normal and pathological levels.

The controls and the corresponding results of the Quality Control Laboratory are stated in the Certificate of Analyses (CoA) added to the kit. The values and ranges stated on the CoA always refer to the current kit lot and must be used for direct comparison of the results.

If available, it is also recommended to make use of national or international Quality Assessment programs in order to ensure the accuracy of the results.

Apply appropriate statistical methods for analyzing control values and trends. If the results of the assay do not agree with the established acceptable ranges of control materials, patient results should be considered invalid.

In this case, please check the following technical areas: Pipetting and timing devices; photometer, expiration dates of reagents, storage and incubation conditions, aspiration and washing methods.

After checking the above-mentioned items without finding any error contact your distributor or DRG directly.

9 PERFORMANCE CHARACTERISTICS

9.1 Specificity of Antibodies (Cross-Reactivity)

The following substances were tested for cross-reactivity of the assay:

Substance	Concentration of Spiked Substance (ng/mL)	Mean Cross-Reactivity (%)
Pro-Hepcidin	1 000	< 0.001
Insulin	100 000	< 0.001
Hepcidin-22	0.61 – 166.6	24.2
Hepcidin-20	6.1 – 166.6	87.7

9.2 Sensitivity

Limit of Blank (LoB)	0.153 ng/mL
Limit of Detection (LoD)	0.304 ng/mL
Limit of Quantification (LoQ)	1.149 ng/mL
Measuring range (range of the assay)	0.153 ng/mL – 81 ng/mL
Linear range	0.475 ng/mL – 81 ng/mL

9.3 Reproducibility

9.3.1 Within-run Precision

The within-run precision for serum was determined with four samples being measured on 8 days in 20 independent runs with 2 determinations each. CV was calculated from 40 determinations.

The within-run precision for plasma was determined with three samples, which were measured with 20 replicates each. CV was calculated from 20 determinations.

Sample	n	Mean (ng/mL)	CV (%)
1 (serum)	40	3.20	3.8
2 (serum)	40	15.47	4.1
3 (serum)	40	21.93	2.2
4 (serum)	40	58.68	0.8

Sample	n	Mean (ng/mL)	CV (%)
1 (heparin plasma)	20	5.78	5.3
2 (citrate plasma)	20	9.48	5.7
3 (EDTA plasma)	20	13.64	9.9

9.3.2 Between-run Precision

The between-run precision was determined with 4 samples covering the complete measuring range within 8 days in 20 independent runs and with 2 replicates per run (40 x 2). CV was calculated from 80 determinations.

Sample	n	Mean (ng/mL)	CV (%)
1	80	3.20	14.4
2	80	15.47	10.5
3	80	21.93	9.5
4	80	58.68	11.2

9.3.3 Between-lot Precision

The inter-assay (between-lots) variation was determined by repeated measurements of 3 samples in 3 different kit lots.

Sample	n	Mean (ng/mL)	CV (%)
1	18	0.95	15.0
2	18	22.67	7.5
3	18	30.38	0.6

9.4 Recovery

Recovery was determined by adding increasing amounts of the analyte to different patient samples containing different amounts of endogenous analyte. The percentage recoveries were determined by comparing expected and measured values of the samples.

	1 Serum	2 Serum	3 Serum	4 Serum	5 EDTA plasma	6 Heparin plasma	7 Citrate plasma
Concentration [ng/mL]	7.60	20.80	27.30	59.20	44.43	37.37	39.95
Average Recovery [%]	104.9	99.6	94.4	98.0	90.4	92.6	81.4
Range of Recovery [%]	from to	98.1 114.5	88.2 111.8	88.4 105.8	87.4 108.7	87.5 97.2	90.5 98.0
							74.1 88.2

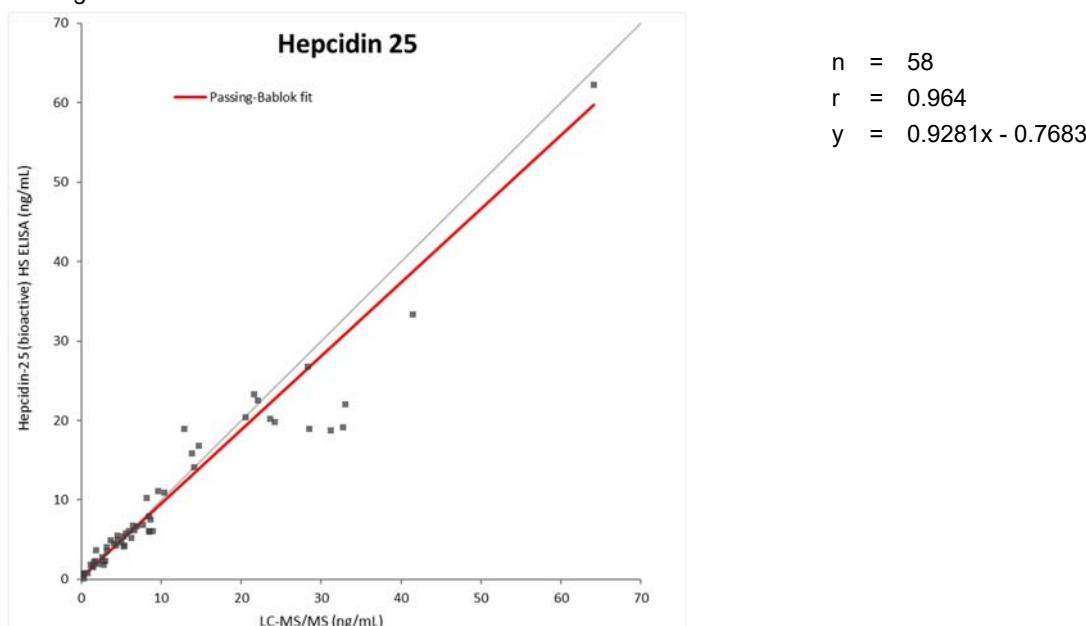
9.5 Linearity

Samples containing different amounts of analyte were serially diluted with *Sample Diluent*. The percentage recovery was calculated by comparing the expected and measured values for the analyte.

	1 Serum	2 Serum	3 Serum	4 Serum	5 EDTA plasma	6 Heparin plasma	7 Citrate plasma
Concentration [ng/mL]	7.60	14.70	27.30	59.00	25.00	25.12	21.14
Average Recovery [%]	98.3	103.1	92.1	98.4	92.0	94.2	95.6
Range of Recovery [%]	from to	96.8 100.0	92.5 114.6	87.9 95.2	86.8 105.8	87.3 95.6	90.0 98.7
							92.8 99.9

9.6 Method Comparison

A comparison of DRG Hepcidin 25 (bioactive) HS ELISA (EIA-5782) (y) and LC-MS measurements (x) using clinical samples gave the following correlation:

10 LIMITATIONS OF THE PROCEDURE

Reliable and reproducible results will be obtained when the assay procedure is performed with a complete understanding of the instructions for use and with adherence to good laboratory practice.

Any improper handling of samples or modification of this test might influence the results.

10.1 Interfering Substances

Hemoglobin (up to 4 mg/mL), bilirubin (up to 0.5 mg/mL) and triglyceride (up to 7.5 mg/mL) have no influence on the assay results. A biotin concentration of up to 1200 ng/mL in a sample has no influence on the assay results.

10.2 Drug Interferences

Until today no substances (drugs) are known to us, which have an influence on the measurement of Hepcidin-25 in a sample.

10.3 High-Dose Hook Effect

A high-dose hook effect is not known for competitive assays.

11 LEGAL ASPECTS

11.1 Reliability of Results

The test must be performed exactly as per the manufacturer's instructions for use. Moreover, the user must strictly adhere to the rules of GLP (Good Laboratory Practice) or other applicable national standards and/or laws. This is especially relevant for the use of control reagents. It is important to always include, within the test procedure, a sufficient number of controls for validating the accuracy and precision of the test.

The test results are valid only if all controls are within the specified ranges and if all other test parameters are also within the given assay specifications. If there is any doubt or concern regarding a result, please contact DRG.

11.2 Therapeutic Consequences

Therapeutic consequences should never be based on laboratory results alone even if all test results are in agreement with the items as stated under point 11.1. Any laboratory result is only a part of the total clinical picture of a patient.

Only in cases where the laboratory results are in acceptable agreement with the overall clinical picture of the patient should therapeutic consequences be derived.

The test result itself should never be the sole determinant for deriving any therapeutic consequences.

11.3 Liability

Any modification of the test kit and/or exchange or mixture of any components of different lots from one test kit to another could negatively affect the intended results and validity of the overall test. Such modification and/or exchanges invalidate any claim for replacement.

Claims submitted due to customer misinterpretation of laboratory results subject to point 11.2 are also invalid. Regardless, in the event of any claim, the manufacturer's liability is not to exceed the value of the test kit. Any damage caused to the test kit during transportation is not subject to the liability of the manufacturer.

11.4 Reporting of Serious Incident

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

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11.4 Notification des incidents graves

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12 LITERATURE

1. Scamuffa N, et al. Regulation of prohepcidin processing and activity by the subtilisin-like proprotein convertases Furin, PC5, PACE4 and PC7. *Gut.* 2008; 57(11):1573-82.
2. Kemna EH, Tjalsma H, Podust VN, Swinkels DW. Mass spectrometry-based hepcidin measurements in serum and urine: analytical aspects and clinical implications. *Clin. Chem.* 2007; 53(4):620-8.
3. Krause A et al. LEAP-1, a novel highly disulfide-bonded human peptide, exhibits antimicrobial activity. *FEBS Lett.* 2000; 480(2-3):147-50.
4. Kroot J et al. Hepcidin in Human Iron Disorders: Diagnostic Implications. *Clin. Chem.* 2011; 57:121650–1669.
5. Nemeth E, Ganz T. The role of hepcidin in iron metabolism. *Acta Haematol.* 2009; 122(2-3):78-86.
6. Hentze MW, Muckenthaler MU, Galy B, Camaschella C. Two to tango: regulation of Mammalian iron metabolism. *Cell.* 2010; 142:24 –38.
7. Swinkels DW, Jorna AT, Raymakers RA. Synopsis of the Dutch multidisciplinary guideline for the diagnosis and treatment of hereditary haemochromatosis. *Neth. J. Med.* 2007; 65(11):452-5.
8. Camaschella C, Silvestri L. Molecular mechanisms regulating hepcidin revealed by hepcidin disorders. *Scientific World Journal.* 2011; 11:1357-66.
9. Girelli D et al. Reduced serum hepcidin levels in patients with chronic hepatitis C. *J. Hepatol.* 2009; 51(5):845-52.
10. Gardenghi S et al. Ineffective erythropoiesis in beta-thalassemia is characterized by increased iron absorption mediated by downregulation of hepcidin and up-regulation of ferroportin. *Blood.* 2007; 109:5027–35.
11. Uehata T et al. Serum hepcidin-25 levels and anemia in non-dialysis chronic kidney disease patients: a cross-sectional study. *Nephrol Dial Transplant.* 2012; 27(3):1076-83.
12. Peeling P, Dawson B, Goodman C, Landers G, Trinder D. Athletic induced iron deficiency: new insights into the role of inflammation, cytokines and hormones. *Eur. J. Appl. Physiol.* 2008; 103: 381–91.

SYMBOLS USED

Symbol	English	Deutsch	Italiano	Español	Français	Português
	European Conformity	CE-Konformitäts-kennzeichnung	Conformità europea	Conformidad europea	Conformité normes européennes	Conformidade Europeia
	Consult instructions for use *	Gebrauchsanweisung beachten *	Consultare le istruzioni per l'uso	Consulte las instrucciones de uso	Consulter la notice d'utilisation	Consultar as instruções de uso
	<i>In vitro</i> diagnostic medical device *	<i>In-vitro</i> -Diagnostikum *	Dispositivo medico-diagnóstico in vitro	Producto sanitario para diagnóstico in vitro	Dispositif médical de diagnostic in vitro	Dispositivo médico para diagnóstico in vitro
	Catalogue number *	Artikelnummer *	No. di Cat.	No de catálogo	Référence	Número de catálogo
	Batch code *	Fertigungslosnummer, Charge *	Lotto no	Número de lote	No. de lot	Código do lote
	Contains sufficient for <n> tests *	Ausreichend für <n> Prüfungen *	Contenuto sufficiente per "n" saggi	Contenido suficiente para <n> ensayos	Contenu suffisant pour "n" tests	Suficiente para <n> determinações
	Temperature limit *	Temperaturbegrenzung *	Temperatura di conservazione	Temperatura de conservacion	Température de conservation	Limites de temperatura
	Use-by date *	Verwendbar bis *	Data di scadenza	Fecha de caducidad	Date limite d'utilisation	Prazo de validade
	Manufacturer *	Hersteller *	Fabbricante	Fabricante	Fabricant	Fabricante
	Date of Manufacture *	Herstellungsdatum *	Data di produzione	Fecha de fabricación	Date de production	Data de fabricação
	Biological risks *	Biologische Risiken *	Rischi biologici	Riesgos biológicos	Risques biologiques	Riscos biológicos
	Caution *	Achtung *	Attenzione	Precaución	Attention	Cuidado
	For research use only	Nur für Forschungszwecke	Solo a scopo di ricerca	Sólo para uso en investigación	Seulement dans le cadre de recherches	
<i>Distributed by</i>	Distributed by	Vertreiber	Distributore	Distribuidor	Distributeur	Distribuidor
<i>Content</i>	Content	Inhalt	Contenuto	Contenido	Conditionnement	Conteúdo
<i>Volume/No.</i>	Volume / No.	Volumen / Anzahl	Volume / Quantità	Volumen / Número	Volume / Quantité	Volume / Quantidade
<i>Microtiterwells</i>	Microtiter plate	Mikrotiterplatte	Piastra per microtitolazione	Placa de microtítulo	Microplaqué	Placa de microtitulação
<i>Antiserum</i>	Antiserum	Antiserum	Antisiero	Antisuero	Antisérum	
<i>Enzyme Conjugate</i>	Enzyme Conjugate	Enzymkonjugat	Coniugato enzimatico	Conjugado enzimático	Conjugué enzymatique	Conjugado Enzimático
<i>Enzyme Complex</i>	Enzyme Complex	Enzymkomplex	Complesso enzimatico	Compuesto enzimático	Complexe enzymatique	Complexo Enzimático
<i>Substrate Solution</i>	Substrate Solution	Substratlösung	Soluzione di substrato	Solución de sustrato	Solution de substrat	Solução de Substrato
<i>Stop Solution</i>	Stop Solution	Stopplösung	Soluzione d' arresto	Solución de parada	Solution d'arrêt	Solução de Paragem
<i>Zero Standard</i>	Zero Standard	Nullstandard	Standard zero	Estándar cero	Zero Standard	Padrão Zero
<i>Standard</i>	Standard	Standard	Standard	Estándar	Standard	Padrão
<i>Control</i>	Control	Kontrolle	Controllo	Control	Contrôle	Controlo
<i>Assay Buffer</i>	Assay Buffer	Assaypuffer	Tampone del dosaggio	Tampón de ensayo	Tampon d'essai	Tampão de Teste
<i>Wash Solution</i>	Wash Solution	Waschlösung	Soluzione di lavaggio	Solución de lavado	Solution de lavage	Solução de Lavagem
<i>1 N NaOH</i>	1 N NaOH	1 N NaOH	1 N NaOH	1 N NaOH	1 N NaOH	1 N NaOH
<i>1 N HCl</i>	1 N HCl	1 N HCl	1 N HCl	1 N HCl	1 N HCl	1 N HCl
<i>Sample Diluent</i>	Sample Diluent	Probenverdünnungsmedium	Diluente per campioni	Diluyente de muestras	Diluant d'échantillon	Diluente de Amostra
<i>Conjugate Diluent</i>	Conjugate Diluent	Konjugatverdünnungsmedium	Diluente coniugato	Diluyente de conjugados	Diluant de conjugué	Diluente de Conjugado

* Definition according to DIN EN ISO 15223-1