User's Manual

SALI-TUBES 500

SALI-TUBES 100

Sampling tubes for collection of saliva samples to be used in the DRG Salivary ELISAs.

SLV-4157; SLV-4158

500

100

DRG 

DRG Instruments GmbH, Germany
Frauenbergstr. 18, D-35039 Marburg
Telefon: +49 (0)6421-1700 0, Fax: +49-(0)6421-1700 50
Internet: www.drg-diagnostics.de
E-Mail: drg@drg-diagnostics.de

DRG 

DRG International, Inc.
USA
Telephone: (908) 233-2079
Fax: (908) 233-0758
E-Mail: corp@drg-international.com

**Please use only the valid version of the package insert provided with the kit.
Verwenden Sie nur die jeweils gültige, im Testkit enthaltene, Arbeitsanleitung.
Si prega di usare la versione valida dell'inserto del pacco a disposizione con il kit.
Por favor, se usa solo la version valida de la metodico técnico incluido aqui en el kit.**

Table of Contents / Inhaltsverzeichnis / Tabla de Contenidos

1	INTENDED USE	2
2	WARNINGS AND PRECAUTIONS.....	2
3	MATERIALS SUPPLIED.....	2
4	SALIVA COLLECTION WITH SALI-TUBES	2
	4.1 Sampling Notes	2
	4.2 Sampling Procedure.....	2
5	PRE-TEST SETUP INSTRUCTIONS	2

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1 INTENDED USE

Sampling tubes for collection of saliva samples to be used in the DRG Salivary Immunoassays

2 WARNINGS AND PRECAUTIONS

1. For in-vitro diagnostic use only. For professional use only.
2. Only to be used in connection with the DRG Saliva tests according to the instructions for use of the test.
3. Do not use other containers without validation. In general, plastic materials tend to adsorb steroids from saliva. The material used for SALI-TUBES does not adsorb steroids in significant amounts.
4. Do not reuse.
5. Used SALI-TUBES have to be treated as hazardous waste according to national biohazard and safety guidelines or regulations.

3 MATERIALS SUPPLIED

SLV-4158	SLV-4157	Component
100*	500*	SALI-TUBE Tubes Ultrapure polypropylene tubes.
100*	500*	Straws Polypropylene.
100*	500*	Label stickers Self-adhesive.

* approximately

4 SALIVA COLLECTION WITH SALI-TUBES

4.1 Sampling Notes

1. The concentration of many analytes in saliva is highly dynamic. Therefore a meaningful timing of the sampling procedure is required.
2. If you want to check the average concentration of steroid hormones in saliva you have to take at least 5 samples within 2 hours. Aliquots of these 5 samples are mixed prior to analysis. Such a sampling strategy will result in reproducible results.
3. Collect Saliva **fasting or earliest 30 minutes after eating or drinking.**
4. Rinsing of the oral cavity with water is recommended just prior to saliva collection (especially after eating).
5. Do not use chewing gum.
6. Do **not use even the slightest red saliva samples** (because of blood contamination).

4.2 Sampling Procedure

1. Label SALI-TUBES with label stickers.
2. Release the saliva with help of the piece of straw into the SALI-TUBE.
3. It is sufficient to fill half of the tube with saliva.

5 PRE-TEST SETUP INSTRUCTIONS

In order to get a clear liquid please deep-freeze the saliva samples prior to use. After complete freezing thaw again, mix and centrifuge. If necessary repeat this procedure several times, until the supernatant is a clear liquid and easy to pipette.

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