## Instructions for Use

# TGF- $\beta$ 2 ELISA

IVD

REF EIA-2396

96

**DRG**

**DRG Instruments GmbH**, Germany  
Frauenbergstraße. 18, D-35039 Marburg  
Phone: +49 (0)6421-1700 0, Fax: +49 (0)6421-1700 50  
Website: [www.drg-diagnostics.de](http://www.drg-diagnostics.de)  
E-mail: [drg@drg-diagnostics.de](mailto:drg@drg-diagnostics.de)

Distributed by:

**DRG**

**DRG International, Inc.**, USA  
841 Mountain Ave., Springfield, NJ 07081  
Phone: (973) 564-7555, Fax: (973) 564-7556  
Website: [www.drg-international.com](http://www.drg-international.com)  
E-mail: [corp@drg-international.com](mailto:corp@drg-international.com)

**Please use only the valid version of the Instructions for Use provided with the kit.  
Verwenden Sie nur die jeweils gültige, im Testkit enthaltene, Gebrauchsanweisung.  
Si prega di usare la versione valida delle istruzioni per l'uso a disposizione con il kit.  
Por favor, se usa solo la version valida de la metodico técnico incluido aqui en el kit.**

**Table of Contents / Inhaltsverzeichnis / Tabella die Contenuti / Tabla de Contenidos**

1	INTRODUCTION .....	2
2	PRINCIPLE OF THE TEST .....	2
3	WARNINGS AND PRECAUTIONS .....	3
4	REAGENTS.....	4
5	SPECIMEN COLLECTION AND PREPARATION.....	6
6	ASSAY PROCEDURE.....	7
7	EXPECTED NORMAL VALUES.....	8
8	QUALITY CONTROL.....	8
9	PERFORMANCE CHARACTERISTICS.....	8
10	LIMITATIONS OF USE.....	8
11	LEGAL ASPECTS .....	9

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## 1 INTRODUCTION

### 1.1 Intended Use

The **DRG TGF- $\beta$ 2 ELISA** is an enzyme immunoassay for the quantitative *in vitro diagnostic* measurement of TGF- $\beta$ 2 in serum, plasma and cell culture supernatant.

### 1.2 Summary and Explanation

Transforming growth factor- $\beta$  (TGF- $\beta$ ) is a multipotent Cytokine with cell- and dose-dependent activities. This molecule is produced by a number of cells and tissue types, e.g. thrombocytes, bone tissue, placenta and kidneys. TGF- $\beta$ 2 is not produced by blood platelets, contrary to TGF- $\beta$ 1. This potent Cytokine modulates embryonic development, bone formation, mammary development, wound healing, hematopoiesis, cell cycle progression and the production of the extracellular matrix. TGF- $\beta$ 2 – null mice were shown to exhibit perinatal mortality and a wide range of developmental defects for a single gene description which include cardiac, lung, craniofacial, limb, spinal column, eye, inner ear and urogenital defects. TGF- $\beta$ 2 has been shown to be a potent growth inhibit factor in the regulation of postnatal cerebellar neurons and neuroblast proliferation. TGF- $\beta$ 2 has been detected in tear fluid. TGF- $\beta$ 2 levels are elevated in the vitreous of patients with proliferative diabetic retinopathy. Elevated plasma levels of TGF- $\beta$ 2 have been described in patients with disseminated malignant melanoma. TGF- $\beta$ 2 concentrations are furthermore elevated in Parkinson's disease in ventricular cerebrospinal fluid.

## 2 PRINCIPLE OF THE TEST

The DRG TGF- $\beta$ 2 ELISA Kit is a solid phase enzyme-linked immunosorbent assay (ELISA) based on the sandwich principle.

Prior to testing the standards and patient samples are diluted in assay buffer, acidified with HCl and then neutralized with Neutralization Buffer.

Afterwards, the neutralized standards and samples are added to the antibody coated (polyclonal) microtiter wells. After the first incubation the unbound sample material is removed by washing. Then a biotinylated mouse anti TGF- $\beta$ 2 antibody and the Streptavidin-HRP Enzyme complex are incubated in succession. An immuno enzyme sandwich complex is formed.

After incubation the unbound conjugate is washed off. Having added the substrate solution, the intensity of colour developed is proportional to the concentration of TGF- $\beta$ 2 in the patient sample.

**3 WARNINGS AND PRECAUTIONS**

1. This kit is for in vitro diagnostic use only. For professional use only.
2. All reagents of this test kit which contain human serum or plasma have been tested and confirmed negative for HIV I/II, HBsAg and HCV by FDA approved procedures. All reagents, however, should be treated as potential biohazards in use and for disposal.
3. Before starting the assay, read the instructions completely and carefully. Use the valid version of instructions for use provided with the kit. Be sure that everything is understood.
4. The microplate contains snap-off strips. Unused wells must be stored at 2 °C - 8 °C in the sealed foil pouch and used in the frame provided.
5. Pipetting of samples and reagents must be done as quickly as possible and in the same sequence for each step.
6. Use reservoirs only for single reagents. This especially applies to the substrate reservoirs. Using a reservoir for dispensing a substrate solution that had previously been used for the conjugate solution may turn solution colored. Do not pour reagents back into vials as reagent contamination may occur.
7. Mix the contents of the microplate wells thoroughly to ensure good test results. Do not reuse microwells.
8. Do not let wells dry during assay; add reagents immediately after completing the rinsing steps.
9. Allow the reagents to reach room temperature (21 °C - 26 °C) before starting the test. Temperature will affect the absorbance readings of the assay. However, values for the patient samples will not be affected.
10. Never pipet by mouth and avoid contact of reagents and specimens with skin and mucous membranes.
11. Do not smoke, eat, drink or apply cosmetics in areas where specimens or kit reagents are handled.
12. Wear disposable latex gloves when handling specimens and reagents. Microbial contamination of reagents or specimens may give false results.
13. Handling should be done in accordance with the procedures defined by an appropriate national biohazard safety guideline or regulation.
14. Do not use reagents beyond expiry date as shown on the kit labels.
15. All indicated volumes have to be performed according to the protocol. Optimal test results are only obtained when using calibrated pipettes and microtiterplate readers.
16. Do not mix or use components from kits with different lot numbers. It is advised not to exchange wells of different plates even of the same lot. The kits may have been shipped or stored under different conditions and the binding characteristics of the plates may result slightly different.
17. Avoid contact with *Stop Solution* containing 0.5 M H<sub>2</sub>SO<sub>4</sub>. It may cause skin irritation and burns.
18. Some reagents contain Proclin 300, BND and/or MIT as preservatives. In case of contact with eyes or skin, flush immediately with water.
19. TMB substrate has an irritant effect on skin and mucosa. In case of possible contact, wash eyes with an abundant volume of water and skin with soap and abundant water. Wash contaminated objects before reusing them. If inhaled, take the person to open air.
20. Chemicals and prepared or used reagents have to be treated as hazardous waste according to the national biohazard safety guideline or regulation.
21. For information on hazardous substances included in the kit please refer to Safety Data Sheets. Safety Data Sheets for this product are available upon request directly from DRG

## 4 REAGENTS

### 4.1 Reagents provided

1. **Microtiterwells**, 12x8 (break apart) strips, 96 wells;  
Wells coated with anti-TGF-β2 antibody (polyclonal).
2. **Standard (Stock Standard)**, 1 vial, 2 mL,  
Concentrations: 1000 pg/mL  
see „Preparation of Reagents“.  
Contain non-mercury preservative.
3. **Assay Buffer, 10X concentrate**, 1 vial, 10 mL,  
Concentrations: 0 pg/mL  
see „Preparation of Reagents“.  
Contain non-mercury preservative.
4. **Enzyme Conjugate**, 1 vial, 11 mL, ready to use,  
Mouse anti TGF antibody conjugated to Biotin.  
Contain non-mercury preservative.
5. **Enzyme Complex**, 1 vial, 11 mL, ready to use  
Streptavidin Peroxidase  
Contain non-mercury preservative.
6. **Substrate Solution**, 1 vial, 14 mL, ready to use,  
Tetramethylbenzidine (TMB).
7. **Stop Solution**, 1 vial, 14 mL, ready to use,  
contains 0.5M H<sub>2</sub>SO<sub>4</sub>,  
Avoid contact with the stop solution. It may cause skin irritations and burns.
8. **Wash Solution**, 1 vial, 30 mL (40X concentrated),  
see „Preparation of Reagents“.
9. **1 M HCl**, 1 vial, 2 mL, ready to use,  
for acidification of the samples.
10. **Neutralization Buffer**, 1 vial, 2 mL, ready to use,  
for neutralization.

**Note:** Additional *Assay Buffer* for sample dilution is available upon request.

### 4.2 Material required but not provided

- 1.5 mL-Reaction Caps (e.g. from Eppendorf) for sample preparation (acidification and neutralization).
- A microtiter plate calibrated reader (450 ± 10 nm) (e.g. the DRG Instruments Microtiter Plate Reader).
- Calibrated variable precision micropipettes.
- Absorbent paper.
- Distilled or deionized water
- Universal indicator paper.
- Timer
- Semi logarithmic graph paper or software for data reduction

### 4.3 Storage Conditions

When stored at 2 °C - 8 °C unopened reagents, except the *Stock Standard*, will retain reactivity until expiration date. Do not use reagents beyond this date.

**Immediately after receipt the *Stock Standard* has to be stored frozen at -20 °C.**

Opened reagents must be stored at 2 °C - 8 °C, or as described in chapter 4.4.

Microtiter wells must be stored at 2 °C - 8 °C. Once the foil bag has been opened, care should be taken to close it tightly again.

Opened kits retain activity for 6 weeks if stored as described above.

#### 4.4 Reagent Preparation

Allow all reagents and required number of strips to reach room temperature prior to use.

##### **Assay Buffer**

Dilute 10 mL of concentrated *Assay Buffer* with 90 mL deionized water to a final volume of 100 mL Working Assay Buffer

##### **Standards**

Serial Dilution of the **Stock Standard** (1,000 pg/mL):

<b>Description</b>	<b>Concentration</b>
<b>Standard A</b>	<b>1,000 pg/mL</b>
<b>Standard B</b> 1 mL Standard A + 1 mL Assay Buffer	<b>500 pg/mL</b>
<b>Standard C</b> 1 mL Standard B + 1 mL Assay Buffer	<b>250 pg/mL</b>
<b>Standard D</b> 1 mL Standard C + 1 mL Assay Buffer	<b>125 pg/mL</b>
<b>Standard E</b> 1 mL Standard D + 1 mL Assay Buffer	<b>62.5 pg/mL</b>
<b>Standard F</b> 1 mL Standard E + 1 mL Assay Buffer	<b>31.25 pg/mL</b>
<b>Standard G</b> 2 mL Assay Buffer	<b>0 pg/mL</b>

**Note:** The diluted standards are stable for 1 week at 2 °C - 8 °C. For longer storage freeze at -20 °C.

##### **Wash Solution**

Add deionized water to the 40X concentrated Wash Solution.

Dilute 30 mL of concentrated *Wash Solution* with 1170 mL deionized water to a final volume of 1200 mL.

*The diluted Wash Solution is stable for 2 weeks at room temperature.*

#### 4.5 Disposal of the Kit

The disposal of the kit must be made according to the national regulations. Special information for this product is given in the Material Safety Data Sheets (see chapter 13).

#### 4.6 Damaged Test Kits

In case of any severe damage to the test kit or components, DRG has to be informed in writing, at the latest, one week after receiving the kit. Severely damaged single components should not be used for a test run. They have to be stored until a final solution has been found. After this, they should be disposed according to the official regulations.

## 5 SPECIMEN COLLECTION AND PREPARATION

Serum or plasma (EDTA- or citrate plasma) and cell culture supernatant can be used in this assay.

Do not use haemolytic, icteric or lipaemic specimens.

Please note: Samples containing sodium azide should not be used in the assay.

### 5.1 Specimen Collection

#### Serum:

Collect blood by venipuncture (e.g. Sarstedt Monovette for serum), allow to clot, and separate serum by centrifugation at room temperature. Do not centrifuge before complete clotting has occurred. Patients receiving anticoagulant therapy may require increased clotting time.

#### Plasma:

Whole blood should be collected into centrifuge tubes containing anti-coagulant (e.g. Sarstedt Monovette with the appropriate plasma preparation) and centrifuged immediately after collection.

### 5.2 Specimen Storage and Preparation

Specimens should be capped and may be stored for up to 24 hours at 2 °C - 8 °C prior to assaying.

Specimens held for a longer time should be frozen only once at -20 °C prior to assay. Thawed samples should be inverted several times prior to testing.

### 5.3 Specimen Dilution

#### 5.3.1 Serum and plasma

Serum and Plasma Samples should be diluted **1:50** with *Assay Buffer* prior to testing.

Please note: The results have to be multiplied with the dilution factor (x 50).

##### Example:

dilution 1:50: 10 µL Serum + 490 µL *Assay Buffer* (mix thoroughly)

#### 5.3.2 Cell Culture Samples

Centrifuge the Cell Culture Samples. Dilute the supernatant with *Assay Buffer*, according to the expected TGF-β2 concentrations, e.g. 1:10, if a high TGF-β2 concentration is expected. The results have to be multiplied with the dilution factor.

##### Example:

dilution 1:10: 10 µL Sample + 90 µL *Assay Buffer* (mix thoroughly)

If in an initial assay, a specimen is found to contain more than the highest standard, the specimens can be further diluted with *Assay Buffer* and reassayed as described in Assay Procedure.

For the calculation of the concentrations this additional dilution factor has to be taken into account.

### 5.4 Acidification and Neutralization of Samples and Standards

1. Add **200 µL Standards** or **pre-diluted sample** into Reaction Caps (e.g. Eppendorf-Caps).

**Please note:** The standards, which have been prepared by serial dilution of the stock standard should also be prepared as described below.

2. Add **20 µL 1 M HCl** to all caps
3. Close caps, mix thoroughly (vortex) and let stand for 15 minutes
4. For Neutralization add **20 µL Neutralization Buffer** to all caps and mix the solution.  
A pH check and correction of pH is not necessary.  
Immediately continue with step 6.2 of assay procedure.

## 6 ASSAY PROCEDURE

### 6.1 General Remarks

- All reagents and specimens must be allowed to come to room temperature before use. All reagents must be mixed without foaming.
- Once the test has been started, all steps should be completed without interruption.
- Use new disposal plastic pipette tips for each standard, control or sample in order to avoid cross contamination.
- Absorbance is a function of the incubation time and temperature. Before starting the assay, it is recommended that all reagents are ready, caps removed, all needed wells secured in holder, etc. This will ensure equal elapsed time for each pipetting step without interruption.
- As a general rule the enzymatic reaction is linearly proportional to time and temperature.

### 6.2 Test Procedure

Each run must include a standard curve.

1. Secure the desired number of Microtiter wells in the frame holder.
2. Dispense **100  $\mu$ L** of each pre-treated **Standard, Control** and **samples with new disposable tips** into appropriate wells.  
(Please refer to chapters "Specimen Dilution" and "Acidification and Neutralization of Samples and Standards".)
3. Cover the plate and incubate 3 hours at room temperature.
4. Briskly shake out the contents of the wells. Rinse the wells **3 times** with diluted wash solution, 300  $\mu$ l per Well. Strike the wells sharply on absorbance paper to remove residual droplets.  
**Important note:** The sensitivity and precision of this assay is markedly influenced by the correct performance of the washing procedure!
5. Dispense **100  $\mu$ l Enzyme Conjugate** into each well.
6. Incubate **120 minutes** at room temperature.
7. Briskly shake out the contents of the wells. Rinse the wells **3 times** with diluted wash solution, 300  $\mu$ l per Well. Strike the wells sharply on absorbance paper to remove residual droplets.
8. Dispense **100  $\mu$ l Enzyme Complex** into each well.
9. Incubate **20 minutes** at room temperature.
10. Briskly shake out the contents of the wells. Rinse the wells **3 times** with diluted wash solution, 300  $\mu$ l per Well. Strike the wells sharply on absorbance paper to remove residual droplets.
11. Add **100  $\mu$ L of Substrate Solution** to each well.
12. Incubate for **10 minutes** at room temperature.
13. Stop the enzymatic reaction by adding **50  $\mu$ L of Stop Solution** to each well.
14. Determine the absorbance (OD) of each well at **450  $\pm$  10 nm** with a microtiter plate reader.  
It is recommended that the wells be read **within 10 minutes** after adding the *Stop Solution*.

### 6.3 Calculation of Results

1. Calculate the average absorbance values for each set of standards, controls and patient samples.
2. Construct a standard curve by plotting the mean absorbance obtained from each standard against its concentration with absorbance value on the vertical(Y) axis and concentration on the horizontal (X) axis.
3. Using the mean absorbance value for each sample determine the corresponding concentration from the standard curve.
4. Automated method: The results in the Instructions for Use have been calculated automatically using a 4-Parameter curve fit. (4 Parameter Rodbard or 4 Parameter Marquardt are the preferred methods.) Other data reduction functions may give slightly different results.
5. The concentration of the samples can be read directly from this standard curve. Multiply the results by the initial dilution factor (for serum serum and plasma samples by 50)  
Samples with concentrations higher than that of the highest standard have to be further diluted. For the calculation of the concentrations this dilution factor has to be taken into account.

## 7 EXPECTED NORMAL VALUES

It is strongly recommended that each laboratory should determine its own normal and abnormal values.

Data can be obtained by request.

## 8 QUALITY CONTROL

Good laboratory practice requires that controls be run with each calibration curve. A statistically significant number of controls should be assayed to establish mean values and acceptable ranges to assure proper performance.

It is recommended to use control samples according to state and federal regulations. The use of control samples is advised to assure the day to day validity of results. Use controls at both normal and pathological levels.

The controls and the corresponding results of the QC-Laboratory are stated in the QC certificate added to the kit. The values and ranges stated on the QC sheet always refer to the current kit lot and should be used for direct comparison of the results.

Employ appropriate statistical methods for analysing control values and trends. If the results of the assay do not fit to the established acceptable ranges of control materials patient results should be considered invalid.

In this case, please check the following technical areas: Pipetting and timing devices; photometer, expiration dates of reagents, storage and incubation conditions, aspiration and washing methods.

After checking the above mentioned items without finding any error contact your distributor or DRG directly.

## 9 PERFORMANCE CHARACTERISTICS

### 9.1 Assay Dynamic Range

The range of the assay is between 10 – 1000 pg/mL.

### 9.2 Specificity of Antibodies (Cross Reactivity)

The following substances were tested for cross reactivity of the assay:

Component	Cross reactivity
TGF-β1	none
TGF-β3	none

### 9.3 Sensitivity

The analytical sensitivity was calculated from the mean plus two standard deviations of twenty (20) replicate analyses of *Standard 0* and was found to be 10 pg/mL.

### 9.4 Reproducibility

Data can be obtained on request.

### 9.5 Recovery

Data can be obtained on request.

### 9.6 Linearity

Data can be obtained on request.

## 10 LIMITATIONS OF USE

Reliable and reproducible results will be obtained when the assay procedure is performed with a complete understanding of the package insert instruction and with adherence to good laboratory practice.

Any improper handling of samples or modification of this test might influence the results.

### 10.1 Interfering Substances

Data can be obtained on request.

### 10.2 Drug Interferences

Until today no substances (drugs) are known to us, which have an influence to the measurement of TGF-β2 in a sample.

### 10.3 High-Dose-Hook Effect

Data can be obtained on request.

## **11 LEGAL ASPECTS**

### **11.1 Reliability of Results**

The test must be performed exactly as per the manufacturer's instructions for use. Moreover the user must strictly adhere to the rules of GLP (Good Laboratory Practice) or other applicable national standards and/or laws. This is especially relevant for the use of control reagents. It is important to always include, within the test procedure, a sufficient number of controls for validating the accuracy and precision of the test.

The test results are valid only if all controls are within the specified ranges and if all other test parameters are also within the given assay specifications. In case of any doubt or concern please contact DRG.

### **11.2 Therapeutic Consequences**

Therapeutic consequences should never be based on laboratory results alone even if all test results are in agreement with the items as stated under point 11.1. Any laboratory result is only a part of the total clinical picture of a patient. Only in cases where the laboratory results are in acceptable agreement with the overall clinical picture of the patient should therapeutic consequences be derived.

The test result itself should never be the sole determinant for deriving any therapeutic consequences.

### **11.3 Liability**

Any modification of the test kit and/or exchange or mixture of any components of different lots from one test kit to another could negatively affect the intended results and validity of the overall test. Such modification and/or exchanges invalidate any claim for replacement.

Claims submitted due to customer misinterpretation of laboratory results subject to point 11.2 are also invalid.

Regardless, in the event of any claim, the manufacturer's liability is not to exceed the value of the test kit. Any damage caused to the test kit during transportation is not subject to the liability of the manufacturer.

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**12 REFERENCES / LITERATURE**

1. Lawrence Da. Transforming growth factor-beta: An overview. *Kidney Int* 1995;47:S19-S23.
2. Roberts AB, Sporn MB. the transforming growth factor-beta. In: Sporn MB, Roberts AB., eds. *Peptide growth factors and their receptors*. Handbook of experimental pharmacology, Heidelberg: Springer Verlag, 1990:419-472.
3. Muller F, Aukrust P, Nilssen DE, Froland SS. Reduced serum level of transforming growth factor-beta in patients with IgA deficiency. *Clin Immunol Immunopathol* 1995;76:203-208.
4. Jiang X, Kanai H, Himomura K, Sawamura M, Yano S. Increased intraplatelet and urinary transforming growth factor-beta in patients with multiple myeloma. *Acta Haematol* 1995;94:1-6.
5. Taketazu F, Miyagawa K, Ichijo H, et al. Decreased level of transforming growth factor beta in blood lymphocytes of patients with aplastic anemia. *Growth Factors* 1992; 6; 85-90.
6. Snowden N, Coupes B, Herrick A, Illingworth K, Jayson MIV, Brenchley PEC. Plasma TGF beta in systemic sclerosis: A cross-sectional study. *Ann Rheum Dis* 1994;53:763-767.

## SYMBOLS USED

Symbol	English	Deutsch	Italiano	Español	Français
	European Conformity	CE-Konformitätskennzeichnung	Conformità europea	Conformidad europea	Conformité normes européennes
	Consult instructions for use *	Gebrauchsanweisung beachten *	Consultare le istruzioni per l'uso	Consulte las instrucciones de uso	Consulter les instructions d'utilisation
	<i>In vitro</i> diagnostic medical device *	<i>In-vitro</i> -Diagnostikum *	Diagnostica in vitro	Diagnóstico in vitro	Diagnostic in vitro
	Catalogue number *	Artikelnummer *	No. di Cat.	No de catálogo	Référence
	Batch code *	Chargencode *	Lotto no	Número de lote	No. de lot
	Contains sufficient for <n> tests *	Ausreichend für <n> Prüfungen *	Contenuto sufficiente per "n" saggi	Contenido suficiente para <n> ensayos	Contenu suffisant pour "n" tests
	Temperature limit *	Temperaturbegrenzung *	Temperatura di conservazione	Temperatura de conservación	Température de conservation
	Use-by date *	Verwendbar bis *	Data di scadenza	Fecha de caducidad	Date limite d'utilisation
	Manufacturer *	Hersteller *	Fabbricante	Fabricante	Fabricant
	Caution *	Achtung *			
	For research use only	Nur für Forschungszwecke	Solo a scopo di ricerca	Sólo para uso en investigación	Seulement dans le cadre de recherches
<i>Distributed by</i>	Distributed by	Vertreiber	Distributore	Distribuidor	Distributeur
<i>Content</i>	Content	Inhalt	Contenuto	Contenido	Conditionnement
<i>Volume/No.</i>	Volume / No.	Volumen / Anzahl	Volume / Quantità	Volumen / Número	Volume / Quantité
<i>Microtiterwells</i>	Microtiterwells	Mikrotiterwells	Micropozzetti	Placas multipocillo	Plaques de micro-titration
<i>Antiserum</i>	Antiserum	Antiserum	Antisiero	Antisuero	Antisérum
<i>Enzyme Conjugate</i>	Enzyme Conjugate	Enzymkonjugat	Tracciante enzimatico	Conjugado enzimático	Conjugué enzymatique
<i>Enzyme Complex</i>	Enzyme Complex	Enzymkomplex	Complesso enzimatico	Complejo enzimático	Complexe enzymatique
<i>Substrate Solution</i>	Substrate Solution	Substratlösung	Soluzione di substrato	Solución de sustrato	Solution substrat
<i>Stop Solution</i>	Stop Solution	Stopplösung	Soluzione d' arresto	Solución de parada	Solution d'arrêt
<i>Zero Standard</i>	Zero Standard	Nullstandard	Standard zero	Estándar cero	Zero Standard
<i>Standard</i>	Standard	Standard	Standard	Estándar	Standard
<i>Control</i>	Control	Kontrolle	Controllo	Control	Contrôle
<i>Assay Buffer</i>	Assay Buffer	Assaypuffer	Tampone del test	Tampón de ensayo	Tampon d'essai
<i>Wash Solution</i>	Wash Solution	Waschlösung	Soluzione di lavaggio	Solución de lavado	Solution de lavage
<i>1N NaOH</i>	1N NaOH	1N NaOH	1N NaOH (idrossido di sodio 1N)	1N NaOH	1N NaOH
<i>1 N HCl</i>	1 N HCl	1 N HCl		1 N HCl	1N HCl
<i>Sample Diluent</i>	Sample Diluent	Probenverdünnungsmedium	Diluyente dei campioni	Solución para dilución de la muestra	Solution pour dilution de l'échantillon
<i>Conjugate Diluent</i>	Conjugate Diluent	Konjugatverdünnungsmedium	Diluyente del tracciante	Solución para dilución del conjugado	Solution pour dilution du conjugué