Instructions for Use

TM-CA 19-9 ELISA

IVD

CE

REF EIA-5069

Σ 96

DRG Instruments GmbH, Germany
Frauenbergstraße. 18, D-35039 Marburg
Phone: +49 (0)6421-1700 0, Fax: +49 (0)6421-1700 50
Website: www.drg-diagnostics.de
E-mail: drg@drg-diagnostics.de

DRG International, Inc., USA
841 Mountain Ave., Springfield, NJ 07081
Phone: (973) 564-7555, Fax: (973) 564-7556
Website: www.drg-international.com
E-mail: corp@drg-international.com

**Please use only the valid version of the Instructions for Use provided with the kit.
 Verwenden Sie nur die jeweils gültige, im Testkit enthaltene, Gebrauchsanweisung.
 Si prega di usare la versione valida delle istruzioni per l'uso a disposizione con il kit.
 Por favor, se usa solo la version valida de la metodico técnico incluido aqui en el kit.
 Utilisez seulement la version valide des Instructions d'utilisation fournies avec le kit.**

Table of Contents / Inhaltsverzeichnis / Tabella die Contenuti / Tabla de Contenidos / Sommaire

1	INTRODUCTION	2	1	INTRODUCCIÓN	22
2	PRINCIPLE OF THE TEST	2	2	FUNDAMENTO DEL ENSAYO	22
3	WARNINGS AND PRECAUTIONS	3	3	PRECAUCIONES.....	22
4	REAGENTS.....	4	4	COMPONENTES DEL KIT.....	23
5	SPECIMEN COLLECTION AND PREPARATION.....	5	5	MUESTRAS	24
6	ASSAY PROCEDURE.....	5	6	PROCEDIMIENTO DE ENSAYO	24
7	EXPECTED NORMAL VALUES.....	7	7	VALORES ESPERADOS	26
8	QUALITY CONTROL.....	7	8	CONTROL DE CALIDAD	26
9	PERFORMANCE CHARACTERISTICS	8	9	CARACTERÍSTICAS DEL ENSAYO	26
10	LIMITATIONS OF USE.....	9	10	LIMITACIONES DE USO	27
11	LEGAL ASPECTS	9	11	ASPECTOS LEGALES	27
1	EINLEITUNG	10	1	INTRODUCCIÓN.....	28
2	TESTPRINZIP	10	2	PRINCIPE DU TEST	28
3	VORSICHTSMAßNAHMEN.....	10	3	PRECAUTIONS D'UTILISATION	28
4	BESTANDTEILE DES KITS	11	4	COMPOSITION DU KIT	29
5	PROBENVORBEREITUNG.....	12	5	ECHANTILLON	30
6	TESTDURCHFÜHRUNG.....	12	6	RÉALISATION DU TEST	30
7	ERWARTETE WERTE	14	7	VALEURS ATTENDUES.....	32
8	QUALITÄTSKONTROLLE	14	8	CONTRÔLE DE QUALITÉ	32
9	ASSAY-CHARACTERISTIKA.....	14	9	CARACTERISTIQUES DU TEST	32
10	GRENZEN DES TESTS	15	10	LIMITES D'UTILISATION.....	33
11	RECHTLICHE GRUNDLAGEN	15	11	ASPECTS LEGAUX.....	33
1	INTRODUZIONE	16	12	REFERENCES / LITERATURE	34
2	PRINCIPIO DEL TEST	16			
3	PRECAUZIONI	16			
4	COMPONENTI DEL KIT	17			
5	CAMPIONI.....	18			
6	ATTUAZIONE DEL TEST	18			
7	VALORI NORMALI	20			
8	CONTROLLO QUALITÀ	20			
9	CARATTERISTICHE DEL TEST	20			
10	LIMITAZIONE DEL TEST	21			
11	ASPETTI LEGALI	21			
				SYMBOLS USED.....	35

1 INTRODUCTION

1.1 Intended Use

The **DRG TM-CA 19-9 ELISA** is a manual enzyme immunoassay for the quantitative measurement of CA 19-9 in human serum.

For *in vitro* diagnostic use only. For laboratory professional use.

1.2 Summary and Explanation

The sialyl Lewis^a epitope is also known as the cancer associated antigen CA 19-9.

CA 19-9 levels are frequently elevated in the serum of patients with cancers of the pancreato-biliary system (i.e. pancreas, gallbladder, biliary tract). In addition, elevated levels of CA 19-9 have been observed in other malignancies such as lung cancer, other gastrointestinal cancers and in some nonmalignant disorders.

It must be taken into account that patients, genotypically negative for the Lewis blood group antigen, will be unable to produce the CA 19-9 antigen even in the presence of malignant tissue.

2 PRINCIPLE OF THE TEST

The DRG TM-CA 19-9 ELISA is a solid phase enzyme-linked immunosorbent assay (ELISA) based on the **sandwich principle**.

The microtiter wells are coated with a monoclonal (mouse) antibody directed towards a unique antigenic site of the CA 19-9 molecule.

During the first incubation, CA 19-9 in the added sample binds to the immobilized antibody. The simultaneously added enzyme conjugate, which contains an anti-CA 19-9 antibody conjugated to horseradish peroxidase, binds to the CA 19-9 forming a sandwich complex.

After a washing step to remove all unbound substances, the solid phase is incubated with the substrate solution. The colorimetric reaction is stopped by addition of stop solution, and optical density (OD) of the resulting yellow product is measured. The intensity of color is proportional to the concentration of the analyte in the sample.

A standard curve is constructed by plotting OD values against concentrations of standards, and concentrations of unknown samples are determined using this standard curve.

3 WARNINGS AND PRECAUTIONS

1. This kit is for in vitro diagnostic use only. For professional use only.
2. All reagents of this test kit which contain human serum or plasma have been tested and confirmed negative for HIV I/II, HBsAg and HCV by FDA approved procedures. All reagents, however, should be treated as potential biohazards in use and for disposal.
3. Before starting the assay, read the instructions completely and carefully. Use the valid version of the package insert provided with the kit. Be sure that everything is understood.
4. The microplate contains snap-off strips. Unused wells must be stored at 2 °C to 8 °C in the sealed foil pouch and used in the frame provided.
5. Pipetting of samples and reagents must be done as quickly as possible and in the same sequence for each step.
6. Use reservoirs only for single reagents. This especially applies to the substrate reservoirs. Using a reservoir for dispensing a substrate solution that had previously been used for the conjugate solution may turn solution colored. Do not pour reagents back into vials as reagent contamination may occur.
7. Mix the contents of the microplate wells thoroughly to ensure good test results. Do not reuse microwells.
8. Do not let wells dry during assay; add reagents immediately after completing the rinsing steps.
9. Allow the reagents to reach room temperature (21 °C to 26 °C) before starting the test. Temperature will affect the absorbance readings of the assay. However, values for the patient samples will not be affected.
10. Never pipet by mouth and avoid contact of reagents and specimens with skin and mucous membranes.
11. Do not smoke, eat, drink or apply cosmetics in areas where specimens or kit reagents are handled.
12. Wear disposable latex gloves when handling specimens and reagents. Microbial contamination of reagents or specimens may give false results.
13. Handling should be done in accordance with the procedures defined by an appropriate national biohazard safety guideline or regulation.
14. Do not use reagents beyond expiry date as shown on the kit labels.
15. All indicated volumes have to be performed according to the protocol. Optimal test results are only obtained when using calibrated pipettes and microtiterplate readers.
16. Do not mix or use components from kits with different lot numbers. It is advised not to exchange wells of different plates even of the same lot. The kits may have been shipped or stored under different conditions and the binding characteristics of the plates may result slightly different.
17. Avoid contact with *Stop Solution* containing 0.5 M H₂SO₄. It may cause skin irritation and burns.
18. Some reagents may contain Proclin 300, BND and/or MIT as preservatives. In case of contact with eyes or skin, flush immediately with water.
19. TMB substrate has an irritant effect on skin and mucosa. In case of possible contact, wash eyes with an abundant volume of water and skin with soap and abundant water. Wash contaminated objects before reusing them. If inhaled, take the person to open air.
20. Chemicals and prepared or used reagents have to be treated as hazardous waste according to the national biohazard safety guideline or regulation.
21. For information on hazardous substances included in the kit please refer to Safety Data Sheets. Safety Data Sheets for this product are available upon request directly from DRG.

4 REAGENTS

4.1 Reagents provided

1. **Microtiterwells**, 12 x 8 strips, 96 wells (break apart);
Wells coated with anti-CA 19-9 antibody (monoclonal).
2. **Zero Standard**, 1 vial, 3 mL, ready to use
Contains non-mercury preservative.
3. **Standard (Standard 1-5)**, 5 vials, 0.5 mL, ready to use;
Concentrations: 15 – 30 – 60 – 120 – 240 U/mL
Contain non-mercury preservative.
4. **Control Low & High**, 2 vials, 0.5 mL each; lyophilized;
For control values and ranges please refer to vial label or to the Certificate of Analysis.
See “Reagent Preparation”.
Contain non-mercury preservative.
5. **Assay Buffer**, 1 vial, 7 mL, ready to use;
Contains non-mercury preservative.
6. **Enzyme Conjugate**, 1 vial, 14 mL, ready to use,
Anti- CA 19-9 antibody conjugated with horseradish peroxidase;
Contains non-mercury preservative.
7. **Substrate Solution**, 1 vial, 14 mL, ready to use,
Contains 3,3',5,5'-tetramethylbenzidine (TMB).
Keep away from direct sun light.
8. **Stop Solution**, 1 vial, 14 mL, ready to use;
Contains 0.5 M H₂SO₄,
Avoid contact with the stop solution. It may cause skin irritations and burns.
9. **Wash Solution**, 1 vial, 30 mL (40X concentrated)
See “Reagent Preparation”.

Note: Additional *Zero Standard* for sample dilution is available upon request.

4.2 Materials required but not provided

- A microtiter plate calibrated reader (450 ± 10 nm)
- Calibrated variable precision micropipettes
- Absorbent paper
- Distilled or deionized water
- Timer
- Graph paper or software for data reduction

4.3 Storage Conditions

When stored at 2 °C to 8 °C unopened reagents will retain reactivity until expiration date. Do not use reagents beyond this date.

Opened reagents must be stored at 2 °C to 8 °C. Microtiter wells must be stored at 2 °C to 8 °C. Once the foil bag has been opened, care should be taken to close it tightly again.

Opened kits retain activity for 8 weeks if stored as described above.

4.4 Reagent Preparation

Bring all reagents and required number of strips to room temperature prior to use.

Control

Reconstitute the lyophilized content with 0.5 mL distilled water and let stand for 10 minutes in minimum. Mix the controls several times before use.

Note: *The reconstituted controls should be apportioned and stored at -20 °C.*

Wash Solution

Add deionized water to the 40X concentrated Wash Solution.

Dilute 30 mL of concentrated *Wash Solution* with 1170 mL distilled water to a final volume of 1200 mL.

The diluted Wash Solution is stable for 1 week at room temperature.

4.5 Disposal of the Kit

The disposal of the kit must be made according to the national regulations. Special information for this product is given in the Safety Data Sheet.

4.6 Damaged Test Kits

In case of any severe damage to the test kit or components, DRG has to be informed in writing, at the latest, one week after receiving the kit. Severely damaged single components should not be used for a test run. They have to be stored until a final solution has been found. After this, they should be disposed according to the official regulations.

5 SPECIMEN COLLECTION AND PREPARATION

Serum must be used in this assay.

Do not use haemolytic, icteric or lipaemic specimens.

Please note: Samples containing sodium azide should not be used in the assay.

5.1 Specimen Collection

Serum:

Collect blood by venipuncture (e.g. Sarstedt Monovette for serum), allow to clot, and separate serum by centrifugation at room temperature. Do not centrifuge before complete clotting has occurred. Patients receiving anticoagulant therapy may require increased clotting time.

5.2 Specimen Storage and Preparation

Specimens should be capped and may be stored for up to 5 days at 2 °C to 8 °C prior to assaying.

Specimens held for a longer time (up to 12 months) should be frozen only once at -20 °C prior to assay. Thawed samples should be inverted several times prior to testing.

5.3 Specimen Dilution

If in an initial assay, a specimen is found to contain more than the highest standard, the specimens can be diluted with *Zero Standard* and reassayed as described in Assay Procedure.

For the calculation of the concentrations this dilution factor has to be taken into account.

Example:

- a) dilution 1:10: 10 µL Serum + 90 µL *Zero Standard* (mix thoroughly)
b) dilution 1:100: 10 µL dilution a) 1:10 + 90 µL *Zero Standard* (mix thoroughly).

6 ASSAY PROCEDURE

6.1 General Remarks

- All reagents and specimens must be allowed to come to room temperature before use. All reagents must be mixed without foaming.
- Once the test has been started, all steps should be completed without interruption.
- Use new disposal plastic pipette tips for each standard, control or sample in order to avoid cross contamination.
- Absorbance is a function of the incubation time and temperature. Before starting the assay, it is recommended that all reagents are ready, caps removed, all needed wells secured in holder, etc. This will ensure equal elapsed time for each pipetting step without interruption.
- As a general rule the enzymatic reaction is linearly proportional to time and temperature.

6.2 Test Procedure

Each run must include a standard curve.

1. Secure the desired number of Microtiter wells in the frame holder.
2. Dispense **50 µL** of each **Standard, Control** and **samples** with new disposable tips into appropriate wells.
3. Dispense **50 µL Assay Buffer** into each well.
4. Incubate for **60 minutes** at room temperature.
5. Briskly shake out the contents of the wells.
Rinse the wells **4 times** with diluted *Wash Solution* (400 µL per well). Strike the wells sharply on absorbent paper to remove residual droplets.
Important note:
The sensitivity and precision of this assay is markedly influenced by the correct performance of the washing procedure!
6. Dispense **100 µL Enzyme Conjugate** into each well.
7. Incubate for **60 minutes** at room temperature.
8. Briskly shake out the contents of the wells.
Rinse the wells **4 times** with diluted *Wash Solution* (400 µL per well). Strike the wells sharply on absorbent paper to remove residual droplets.
9. Add **100 µL of Substrate Solution** to each well.
10. Incubate for **30 minutes** at room temperature.
11. Stop the enzymatic reaction by adding **100 µL of Stop Solution** to each well.
12. Determine the absorbance (OD) of each well at **450 ± 10 nm** with a microtiter plate reader.
It is recommended that the wells be read **within 10 minutes** after adding the *Stop Solution*.

6.3 Calculation of Results

1. Calculate the average absorbance values for each set of standards, controls and patient samples.
2. Manual method: Using linear graph paper, construct a standard curve by plotting the mean absorbance obtained from each standard against its concentration with absorbance value on the vertical (Y) axis and concentration on the horizontal (X) axis.
3. Using the mean absorbance value for each sample determine the corresponding concentration from the standard curve.
4. Automated method: The results in the Instructions for Use have been calculated automatically using a 4-Parameter curve fit. (4 Parameter Rodbard or 4 Parameter Marquardt are the preferred methods.) Other data reduction functions may give slightly different results.
5. The concentration of the samples can be read directly from this standard curve. Samples with concentrations higher than that of the highest standard have to be further diluted or reported as > 240 U/mL. For the calculation of the concentrations this dilution factor has to be taken into account.

6.3.1 Example of Typical Standard Curve

The following data is for demonstration only and **cannot** be used in place of data generations at the time of assay.

Standard	Optical Units (450 nm)
Standard 0 (0 U/mL)	0.04
Standard 1 (15 U/mL)	0.26
Standard 2 (30 U/mL)	0.44
Standard 3 (60 U/mL)	0.78
Standard 4 (120 U/mL)	1.30
Standard 5 (240 U/mL)	1.99

7 EXPECTED NORMAL VALUES

It is strongly recommended that each laboratory should determine its own normal and abnormal values.

In a study conducted with apparently normal healthy adults, using the DRG TM-CA 19-9 ELISA the following values are observed:

Population	n	Mean (U/mL)	Median (U/mL)	5 th - 95 th Percentile (U/mL)	2.5 th - 97.5 th Percentile (U/mL)	Range (min. - max.) (U/mL)
Males and females	71	8.08	5.70	1.15 - 20.90	0.27 - 26.15	0.04 - 28.70

Several studies recommended a cut-off value of 37 U/mL to differentiate between invasive and benign intraductal papillary mucinous neoplasm (IPMN) in chronic pancreatitis (10, 12, 13). Other studies suggest a value of 70 U/mL – 75 U/mL for differentiating benign from malignant pathology (14, 15).

The results alone should not be the only reason for any therapeutic consequences. The results should be correlated to other clinical observations and diagnostic tests.

8 QUALITY CONTROL

Good laboratory practice requires that controls be run with each calibration curve. A statistically significant number of controls should be assayed to establish mean values and acceptable ranges to assure proper performance.

It is recommended to use control samples according to state and federal regulations. The use of control samples is advised to assure the day to day validity of results. Use controls at both normal and pathological levels.

The controls and the corresponding results of the QC-Laboratory are stated in the QC certificate added to the kit. The values and ranges stated on the QC sheet always refer to the current kit lot and should be used for direct comparison of the results.

It is also recommended to make use of national or international Quality Assessment programs in order to ensure the accuracy of the results.

Employ appropriate statistical methods for analysing control values and trends. If the results of the assay do not fit to the established acceptable ranges of control materials patient results should be considered invalid.

In this case, please check the following technical areas: Pipetting and timing devices; photometer, expiration dates of reagents, storage and incubation conditions, aspiration and washing methods.

After checking the above mentioned items without finding any error contact your distributor or DRG directly.

9 PERFORMANCE CHARACTERISTICS

9.1 Detection capability

The Limit of Blank (LoB) is 0.12 U/mL.

The range of the assay is between 0.12 U/mL – 240 U/mL.

9.2 Specificity of Antibodies (Cross Reactivity)

No cross-reactivity was observed for AFP, CA-125, CEA, Cyr61, HCG, HPL, L1-CAM, and FSH.

Cross reactivity with CA 72-4 and CA 15-3 is most likely due to samples, which contained small contamination of CA 19-9.

Until today no substances (drugs) are known to us, which have an influence on the measurement of CA 19-9 in a sample. Please note that heavy consumption of black tea may lead to increased values for CA 19-9 in serum [16].

9.3 Precision

9.3.1 Repeatability (within-run)

The within assay variability is shown below:

Sample	n	Mean (U/mL)	CV (%)
1	20	27.7	8.4
2	20	47.7	9.2
3	20	74.0	9.6

9.3.2 Reproducibility (between-run)

The between assay variability is shown below:

Sample	n	Mean (U/mL)	CV (%)
1	39	5.7	12.2
2	40	10.7	8.2
3	40	11.4	10.5

9.3.3 Reproducibility (within-lot)

The inter-assay (between-lots) variation was determined by measuring each sample 6 times with 3 different kit lots (n = 18):

Sample	n	Mean (U/mL)	CV (%)
1	18	10.5	10.1
2	18	14.3	7.8
3	18	25.2	3.4
4	18	28.6	1.0

9.4 Recovery

Samples have been spiked by adding CA 19-9 solutions with known concentrations.

The % recovery was calculated by multiplying the ratio of the measured and expected values with 100.

	Sample 1	Sample 2	Sample 3
Concentration [U/mL]	17.60	22.10	22.60
Average Recovery [%]	91.1	87.7	91.9
Range of Recovery [%]	from	87.2	85.5
	to	98.5	91.6
		87.4	95.9

9.5 Linearity

Samples were measured undiluted and in serial dilutions from 1:2 to 1:16 with Zero Standard. The recovery (%) was calculated by multiplying the ratio of expected and measured values with 100.

		Sample 1	Sample 2	Sample 3
Concentration [U/mL]		25.80	41.70	82.70
Average Recovery [%]		109.7	102.6	100.8
Range of Recovery [%]	from	103.9	97.8	93.1
	to	114.7	104.6	108.5

10 LIMITATIONS OF USE

Reliable and reproducible results will be obtained when the assay procedure is performed with a complete understanding of the package insert instruction and with adherence to good laboratory practice.

Any improper handling of samples or modification of this test might influence the results.

10.1 Interfering Substances

Haemoglobin (up to 4 mg/mL), Bilirubin (up to 0.5 mg/mL) and Triglyceride (up to 30 mg/mL) have no influence on the assay results.

The assay contains reagents to minimize interference of HAMA and heterophilic antibodies. However, extremely high titers of HAMA or heterophilic antibodies may interfere with the test results.

10.2 Drug Interferences

Until today no substances (drugs) are known to us, which have an influence to the measurement of CA 19-9 in a sample.

10.3 High-Dose-Hook Effect

Hook effect was not observed in this test up to a concentration of 15,360 U/mL of CA 19-9.

11 LEGAL ASPECTS

11.1 Reliability of Results

The test must be performed exactly as per the manufacturer's instructions for use. Moreover the user must strictly adhere to the rules of GLP (Good Laboratory Practice) or other applicable national standards and/or laws. This is especially relevant for the use of control reagents. It is important to always include, within the test procedure, a sufficient number of controls for validating the accuracy and precision of the test.

The test results are valid only if all controls are within the specified ranges and if all other test parameters are also within the given assay specifications. In case of any doubt or concern please contact DRG.

11.2 Therapeutic Consequences

Therapeutic consequences should never be based on laboratory results alone even if all test results are in agreement with the items as stated under point 11.1. Any laboratory result is only a part of the total clinical picture of a patient. Only in cases where the laboratory results are in acceptable agreement with the overall clinical picture of the patient should therapeutic consequences be derived.

The test result itself should never be the sole determinant for deriving any therapeutic consequences.

11.3 Liability

Any modification of the test kit and/or exchange or mixture of any components of different lots from one test kit to another could negatively affect the intended results and validity of the overall test. Such modification and/or exchanges invalidate any claim for replacement.

Claims submitted due to customer misinterpretation of laboratory results subject to point 11.2 are also invalid.

Regardless, in the event of any claim, the manufacturer's liability is not to exceed the value of the test kit. Any damage caused to the test kit during transportation is not subject to the liability of the manufacturer.

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12 REFERENCES / LITERATURE

1. Koprowski H, Steplewski Z, Mitchell K, Herlyn M, Herlyn D, Fuhrer P: Colorectal carcinoma antigens detected by hybridoma antibodies. *Somatic Cell Genetics* 5:957-972, 1979.
2. Koprowski H, Herlyn M, Steplewski Z, Sears H: Specific antigen in serum of patients with colon carcinoma. *Science* 212:53-55, 1981.
3. Magnani J, et al.: The Antigen of Tumor-Specific Monoclonal Antibody is a Ganglioside Containing Sialylated Lacto-N-Fucopentaose II. *Federation Proceedings*, 41:898, 1982.
4. Del Villano B, Brennan S, Brock P, Bucher C, Liu V, McClure M, Rake B, Space S, Westrick B, Schoemaker H, Zurawski V Jr: Radioimmunoassay for a Monoclonal Antibody-Defined Tumor Marker, CA 19-9. *Clin Chem* 29:549-552, 1983.
5. Steinberg W, Gelfand R, Anderson K, Glenn J, Kurtzman SH, Sindelar W, Toskes P: Comparison of the Sensitivity and Specificity of the CA 19-9 and Carcinoembryonic Antigen Assays in Detecting Cancer of the Pancreas. *Gastroenterology* 90:343-349, 1986.
6. Ritts R Jr, Del Villano B, Go VLW, Herberman R, Klug T, Zurawski V Jr: Initial Clinical Evaluation of an Immunoradiometric Assay for CA 19-9 Using the NCI Serum Bank. *Int J Cancer* 33:339-345, 1984.
7. Safi F, Beger H, Bittner R, Buchler M, Krautzberger W: CA 19-9 and pancreatic adenocarcinoma. *Cancer* 57:779, 1986.
8. NCCLS Guideline EP7-P, entitled "Interference Testing in Clinical Chemistry; Proposed Guideline" August 1986.
9. NCCLS Guideline EP5-A, entitled "Evaluation of Precision Performance of Clinical Chemistry Devices; Approved Guideline," February 1999.
10. Del Villano B and Zurawski V Jr: The carbohydrate antigenic determinant 19-9 (CA 19-9): a monoclonal antibody defined tumor marker. *Immunodiagnosics – Laboratory and Research Methods in Biology and Medicine, Volume 8*. Ed. By J. Hyun and R. Aloisi, Alan R. Liss, New York, pp. 269-282, 1983.
11. Dietel M, Arps H, Müller-Hagen S, Sieck M and Hoffmann L: Antigen detection by the monoclonal antibodies CA 19-9 and CA 125 in normal and tumor tissue and patients' sera – *J Cancer Research Clinical Oncology* 111:257-265, 1986
12. Fritz S, Hackert T, Hinz U, Hartwig W, Büchler MW, Werner J.: Role of serum carbohydrate antigen 19-9 and carcinoembryonic antigen in distinguishing between benign and invasive intraductal papillary mucinous neoplasm of the pancreas. *Br J Surg.* 98(1):104-10, 2011.
13. Bedi MM, Gandhi MD, Jacob G, Lekha V, Venugopal A, Ramesh H: CA 19-9 to differentiate benign and malignant masses in chronic pancreatitis: is there any benefit? *Indian J Gastroenterol* 28(1):24-7, 2009.
14. Morris-Stiff G, Teli M, Jardine N, Puntis MC: CA19-9 antigen levels can distinguish between benign and malignant pancreaticobiliary disease. *Hepatobiliary Pancreat Dis Int* 8(6):620-6, 2008.
15. Steinberg WM, Gelfand R, Anderson KK, Glenn J, Kurtzman SH, Sindelar WF, Toskes PP: Comparison of the sensitivity and specificity of the CA19-9 and carcinoembryonic antigen assays in detecting cancer of the pancreas. *Gastroenterology* 90(2):343-9, 1986.
16. Howaizi M, Abboura M, Krespine C, et al: A new cause for CA19.9 elevation: heavy tea consumption. *Gut* 2003; 52:913-914.

SYMBOLS USED

Symbol	English	Deutsch	Italiano	Español	Français
	European Conformity	CE-Konformitätskennzeichnung	Conformità europea	Conformidad europea	Conformité normes européennes
	Consult instructions for use *	Gebrauchsanweisung beachten *	Consultare le istruzioni per l'uso	Consulte las instrucciones de uso	Consulter les instructions d'utilisation
	In vitro diagnostic medical device *	In-vitro-Diagnostikum *	Diagnostica in vitro	Diagnóstico in vitro	Diagnostic in vitro
	Catalogue number *	Artikelnummer *	No. di Cat.	No de catálogo	Référence
	Batch code *	Chargencode *	Lotto no	Número de lote	No. de lot
	Contains sufficient for <n> tests *	Ausreichend für <n> Prüfungen *	Contenuto sufficiente per "n" saggi	Contenido suficiente para <n> ensayos	Contenu suffisant pour "n" tests
	Temperature limit *	Temperaturbegrenzung *	Temperatura di conservazione	Temperatura de conservacion	Température de conservation
	Use-by date *	Verwendbar bis *	Data di scadenza	Fecha de caducidad	Date limite d'utilisation
	Manufacturer *	Hersteller *	Fabbricante	Fabricante	Fabricant
	Caution *	Achtung *	Attenzione	Precaución	Attention *
	For research use only	Nur für Forschungszwecke	Solo a scopo di ricerca	Sólo para uso en investigación	Seulement dans le cadre de recherches
	Distributed by	Vertreiber	Distributore	Distribuidor	Distributeur
<i>Content</i>	Content	Inhalt	Contenuto	Contenido	Conditionnement
<i>Volume/No.</i>	Volume / No.	Volumen/Anzahl	Volume/Quantità	Volumen/Número	Volume/Quantité
<i>Microtiterwells</i>	Microtiter wells	Mikrotiterwells	Micropozzetti	Placas multipocillo	Plaques de microtitration
<i>Antiserum</i>	Antiserum	Antiserum	Antisiero	Antisero	Antisérum
<i>Enzyme Conjugate</i>	Enzyme Conjugate	Enzymkonjugat	Tracciante enzimatico	Conjugado enzimático	Conjugué enzymatique
<i>Enzyme Complex</i>	Enzyme Complex	Enzymkomplex	Complesso enzimatico	Complex enzimático	Complexe enzymatique
<i>Substrate Solution</i>	Substrate Solution	Substratlösung	Soluzione di substrato	Solución de sustrato	Solution substrat
<i>Stop Solution</i>	Stop Solution	Stopplösung	Soluzione d'arresto	Solución de parada	Solution d'arrêt
<i>Zero Standard</i>	Zero Standard	Nullstandard	Standard zero	Estándar cero	Zero Standard
<i>Standard</i>	Standard	Standard	Standard	Estándar	Standard
<i>Control</i>	Control	Kontrolle	Controllo	Control	Contrôle
<i>Assay Buffer</i>	Assay Buffer	Assaypuffer	Tampone del test	Tampón de ensayo	Tampon d'essai
<i>Wash Solution</i>	Wash Solution	Waschlösung	Soluzione di lavaggio	Solución de lavado	Solution de lavage
<i>1N NaOH</i>	1N NaOH	1N NaOH	1N NaOH (idrossido di sodio 1N)	1N NaOH	1N NaOH
<i>1 N HCl</i>	1 N HCl	1 N HCl	1 N HCl	1 N HCl	1N HCl
<i>Sample Diluent</i>	Sample Diluent	Probenverdünnungsmedium	Diluyente dei campioni	Solución para dilución de la muestra	Solution pour dilution de l'échantillon
<i>Conjugate Diluent</i>	Conjugate Diluent	Konjugatverdünnungsmedium	Diluyente del tracciante	Solución para dilución del conjugado	Solution pour dilution du conjugué