## Instructions for Use

# Fecal Extraction System 2.6

This product must be used exclusively with the DRG:HYBRiD-XL Calprotectin test (REF: HYE-5767).

IVD

REF HYI-5989

 100 tubes

**DRG** 

DRG Instruments GmbH, Germany  
Frauenbergstraße. 18, D-35039 Marburg  
Phone: +49 (0)6421-1700 0, Fax: +49 (0)6421-1700 50  
Website: [www.drg-diagnostics.de](http://www.drg-diagnostics.de)  
E-mail: [drg@drg-diagnostics.de](mailto:drg@drg-diagnostics.de)

Distributed by:

**DRG** 

DRG International, Inc., USA  
841 Mountain Ave., Springfield, NJ 07081  
Phone: (973) 564-7555, Fax: (973) 564-7556  
Website: [www.drg-international.com](http://www.drg-international.com)  
E-mail: [corp@drg-international.com](mailto:corp@drg-international.com)

**Please use only the valid version of the Instructions for Use provided with the kit.  
 Verwenden Sie nur die jeweils gültige, im Testkit enthaltene, Gebrauchsanweisung.  
 Utilizzare esclusivamente la versione valida delle Istruzioni per l'uso in dotazione con il kit.  
 Por favor, utilizar solo la versión válida de las instrucciones de uso que se incluyen en el kit.  
 Veuillez n'utiliser que la version valide du mode d'emploi fournie avec les kits.  
 Używać aktualnej wersji instrukcji dołączonej do zestawu.**

### Contents / Inhaltsverzeichnis / Contenuti / Contenido / Contenu / Spis treści

1	INTENDED USE.....	2
2	REAGENTS .....	2
3	SPECIMEN COLLECTION.....	2
1	VERWENDUNGSZWECK.....	4
2	REAGENZIE.....	4
3	PROBENSAMMLUNG .....	4
1	CAMPO DI APPLICAZIONE .....	6
2	REAGENTI.....	6
3	RACCOLTA DEI CAMPIONI .....	6
1	USO PREVISTO .....	8
2	REACTIVOS .....	8
3	COLECCIÓN DE MUESTRAS .....	8
1	PRZEZNACZENIE .....	10
2	ODCZYNNIKI .....	10
3	ZBIERANIE PRÓBEK .....	10
	SYMBOLS USED .....	12

## 1 INTENDED USE

This **Fecal Extraction System 2.6** is intended for the preparation of stool samples to be used in the **DRG:HYBRiD-XL Calprotectin** test (REF: HYE-5767).

For in vitro diagnostic use only.

## 2 REAGENTS

100 tubes, (white dipstick)	each containing 1.5 mL extraction buffer, ready to use; colorless liquid, contains non-mercury preservative.
--------------------------------	--

### 2.1 Storage Conditions

The tubes have to be stored at 2 °C - 8 °C.


When stored at 2 °C to 8 °C, unopened tubes will retain reactivity until expiration date.

## 3 SPECIMEN COLLECTION

- The raw stool sample has to be thawed. For particularly heterogeneous samples we recommend a mechanical homogenization using an applicator, inoculation loop or similar device.
- Allow the extraction buffer to reach room temperature.
- Unscrew the white dipstick and remove from the specimen tube (FES). Insert white dipstick into sample at 3 different points. The lower part of the dipstick has notches which need to be covered completely with stool after inserting it into the sample. Place dipstick back into the tube. When putting the stick back into the tube, excess material will be stripped off, leaving 7.5 mg of sample to be diluted. Screw tightly to close the tube.
- Shake the tube well until no stool sample remains in the notches.

**Important:** Please make sure that you have a maximally homogenous suspension after shaking. Especially with more solid samples, soaking the sample in the tube with buffer for app. 10 minutes improves the result.

- Allow sample to stand for approx. 10 minutes until sediment has settled. Floating material like shells of grains can be neglected.
- Carefully unscrew the complete cap of the tube including the blue ring plus the dipstick from the FES. Discard cap and dipstick. Make sure that the sediment will not be dispersed again.

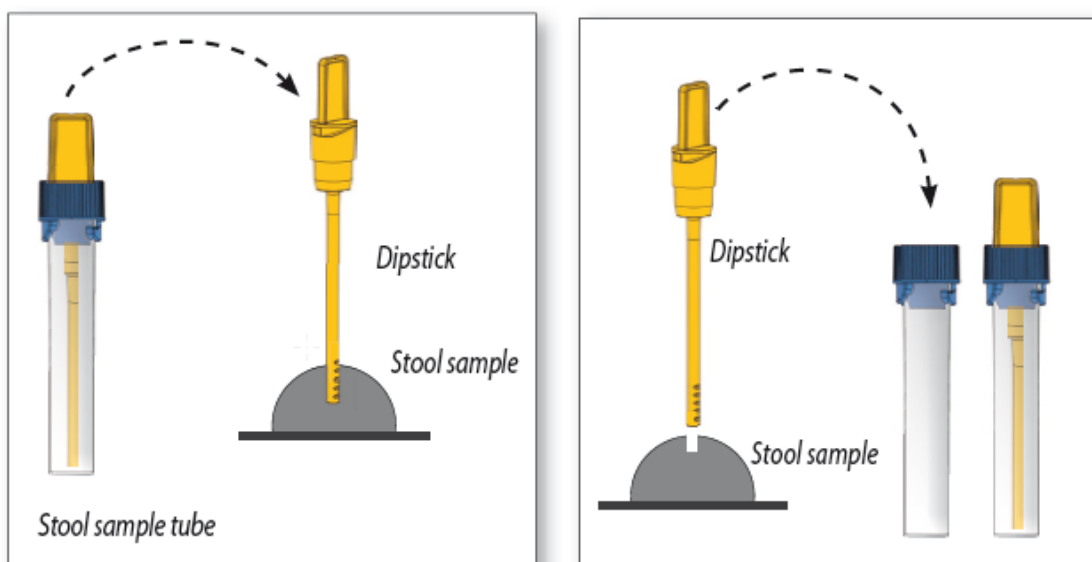**Solid stool samples**

<b>Manual Dilution in the FES</b>	<b>1:200</b>
1 g stool is equivalent to 1 mL	
Applied amount of stool with dipstick*:	7.5 mg
Extraction Buffer Volume in the FES:	1.5 mL
Dilution Factor:	1:200

- \* If the sample volume is too low to completely immerse the notches of the dipstick, please transfer (weigh) 7.5 mg of stool in a separate vial (recommended: 2.0 mL polypropylene reaction tube) and transfer 1.5 mL of extraction buffer to this separate vial and vortex well.

**Liquid stool samples**

For liquid stool samples do not use the dipstick, but pipette the stool sample directly in the extraction tube.

<b>Manual Dilution in the FES</b>	<b>1:200</b>
1 µL stool is equivalent to 1 mg stool	
Applied amount of stool:	7.5 µL
Extraction Buffer Volume in the FES:	1.5 mL
Dilution Factor:	1:200

- The sample now is ready for use in the **DRG:HYBRiD-XL Calprotectin** test.

**Place the opened extraction tube into the adapter for the primary arc.**

**Please note!**

The dilution factor 200 due to the pretreatment of the stool samples in the FES tubes has **not** to be taken into account. This factor is included in the automatic calculation of the analyzer.

**NOTE:**

**For further details please refer to the Instructions for Use of the DRG:HYBRiD-XL Calprotectin test, chapter 5 SPECIMEN COLLECTION AND PREPARATION !!**

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